

# WHY WAIT for test results?



## FRENDS™ System



FAST



EASY TO USE



ACCURATE

# tPSA

Rapid quantitative assay  
for total Prostate Specific Antigen

- Improve patient care
- Increase patient satisfaction
- Increase practice efficiency
- Increase practice profitability

# FREND™ System

Accurate, automated testing procedure – simple 3-step operation  
No need to batch your sample – run when you are ready

- Controls: Minimum once a month two level liquid controls
- Electronic code chip calibration – no need to run liquid
- Daily electronic system QC control – takes only 1.5 minutes
- Stores 500 Test results



## Test procedure




# FREND™ PSA Plus

## Specification

Analytical measurement range	0.08 ~ 25 ng/mL
Sample type	Plasma or Serum
Sample volume	35 µL
Limit of detection	0.08 ng/mL
Reaction time	Max 4 min.
1 Box	25 tests
Storage condition	2°C – 8 °C (35°F - 46°F)

\* The specifications are subject to change without prior notice.

## Correlation graph

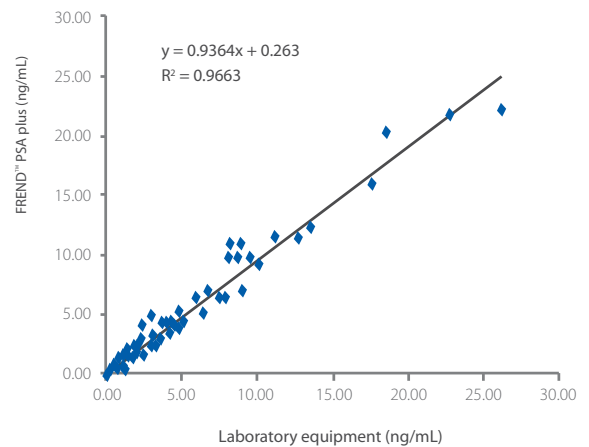

The  $R^2$  correlation value of FREND™ PSA plus is more than 0.96 against laboratory results.

# Ordering information

FREND™ System	Cat# F10
FREND™ PSA Plus	Cat# FRPS 025



website [www.nanoentek.com](http://www.nanoentek.com)  
e-mail [ivdst@nanoentek.com](mailto:ivdst@nanoentek.com)

NESCT-FRPS-002EN (V.0.2)

### NanoEntek, Inc.

#### Head Office

12F, 5, Digital-ro 26-gil, Guro-gu, Seoul, 08389, Korea  
Tel +82-2-6220-7940 / Fax +82-2-6220-7999

#### NanoEntek America, Inc.

220 Bear Hill Road, Suite 102, Waltham, MA 02451, USA  
Tel +1-781-472-2558 / Tel CS: +1-781-216-8916 / Fax: +1-781-790-5649

#### NanoEntek Europe I med-tech supplies GmbH

Lochamerstr. 4a, 82152 Martinsried, Germany  
Tel +49-89-21-55-38-43 / Fax +49-89-99-95-46-60