Actim[®] PROM

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1. Erdemoglu E and Mungan T, Significance of detecting insulin-like growth factor binding protein-1 in cervicovaginal secretions: Comparison with nitrazine test and amniotic fluid volume assessment, Acta Obstet Gynecol Scand (2004) 83: 622-626.
2. Gaucherand P et al, Comparative study of three vaginal markers of the premature rupture of membranes, Acta Obstet Gynecol Scand (1997) 76:536 - 540.
3. Guibourdenche J et al, Rapid detection of insulin-like growth factor-binding protein-1 and foetal fibronectin in cervico-vaginal secretions to diagnose premature membrane rupture, Ann Clin Biochem (1999) 36:388 - 390.
4. Jain K and Morris PG, A clinical study to evaluate the usefulness of the MAST test in diagnosing pre-labour rupture of membranes, J Obstet Gynaecol (1998) 18:33 - 36.
5. Kubota T and Takeuchi H, Evaluation of insulin-like growth factor binding protein-1 as a diagnostic tool for rupture of the membranes, J Obstet Gynecol Res (1998) 24: 411 - 417.
6. Palacio et al, BMC Pregnancy and Childbirth 2014, 14:183
7. Ragosch V et al, Insulin like growth factor binding protein 1 (IGFBP-1) und fetales Fibronectin in der Diagnostik eines vorzeitigen Blasensprunges, GebFra (1996) 56:1 - 6.
8. Rutanen E-M et al, Evaluation of a rapid strip test for insulin-like growth factor binding protein-1 in the diagnosis of ruptured fetal membranes, Clinica Chimica Acta (1996) 253: 91-101.
9. Rutanen E-M et al, Measurement of insulin-like growth factor binding protein-1 in cervical/vaginal secretions: comparison with the ROMcheck Membrane immunoassay in the diagnosis of ruptured fetal membranes, Clinica Chimica Acta (1993) 214: 73 - 81.
10. Rutanen E-M, Insulin-like growth factors in obstetrics, Curr Opin Obstet Gynecol (2000) 12:163 - 168.
11. Walthen NC et al, Levels of insulin-like growth factor-binding protein-1 increase rapidly in amniotic fluid from 11 to 16 weeks of pregnancy, J Endocrinol (1993) 137:R1 - R4.
12. Yang J et al, Vaginal bleeding during pregnancy and preterm birth, Am J Epidemiol (2004) 160:118 - 125.

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